HENRY SCHEIN®

(GB)

Impression Tray Resin LC

Instruction for Use

Impression Tray Resin LC is a resin in the form of contoured plates for maxillae and mandibles. It is used for the fabrication of individual trays for crown & bridges and in a total prosthetic technique. It can also be used for the fabrication of bases for bite patterns and odontograms. The tray resin is ready for use, easy to handle and has a pleasant, peppermint smell. ***Impression Tray Resin LC*** can be cured under room and daylight (lengthy procedure), but cures quickly under halogen and UVA light. The individual trays can be used directly after fabrication.

Method of Use:

Allow ***Impression Tray Resin LC*** to reach room temperature before use. Mark margins for the individual tray on the plaster model. If necessary, use a space maintainer. If space maintainer is not necessary, insulate plaster model. Remove protection foil (close the lid of the box immediately) and mould plate onto the model and cut the desired rim. The transparency of ***Impression Tray Resin LC*** allows very exact modelling. The excess material can easily be used for making a handle.

Impression Tray Resin LC is then polymerized on the model in a standard light unit (halogen or UVA light). Depending on the thickness of the contoured tray, polymerize for 4 - 5 min. Longer polymerization will improve the physical properties of the material. The custom tray is then ready for use immediately. A slight inhibition film, caused by the oxygen in the air, can be removed with a tissue moistened with alcohol.

Storage: use – Store in cool place and well protected from light!

Precautions/ Indications:

Too short exposure or too low intensity of exposure causes a loss of stability. Impression Tray Resin LC contains BIS-GMA, UDMA, TEGDMA, peppermint oil, BHT. Do not use in cases where known allergies against these ingredients exist.

Packaging:

Impression Tray Resin LC is available in different shades in both upper or lower forms. Each pack contains 50 plates.

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Henry Schein U.K. Holdings Ltd.